

RESCOMF®

Destroys
>99.99%
of germs

Ventilation
Disinfector
XD-100

CE | FC | 

**Ventilation Disinfector:
Thorough cleaning
& sanitizing system**

Long-term using of a ventilator will lead to bacteria and mold accumulation in their CPAP/Bi-Level system, causing various health complications.



Brand	RESCOMF
Item Name	Ventilation Disinfecter
Model	XD100
Weight	205g
Rechargeable source	2000mAh
Rated Power	≤15VA
Storage Conditions	-20°C ~ 55°C
Operational environment	-5°C ~ 45°C
IP Rating	IPX1

|| Dimension ||





Comparing to other brands, there are 8 advantages in XD100.

- 1 Small and lightweight, easy to carry.
- 2 Effective disinfection.
- 3 One-button control, no need setting.
- 4 Connect to most CPAP models.
- 5 Maintenance free, no consumables.
- 6 Ultra quiet.
- 7 Long battery life.
- 8 Rechargeable lithium-ion battery. Charger included.



Standard tubing port for various brand of CPAP Machine

Disinfection Method



Operating your Ventilation Disinfecter

1. To turn on, press and hold the On/Off button for 3 seconds. The buzzer will beep once, the light will turn green and ozone will start streaming.
 2. Ozone will stream for 30 minutes, then the device will circulate fresh air for 5 minutes. The last 5 minutes of fresh air will remove most of the residual ozone.
 3. After the 35 minutes cycle has been completed, the machine will automatically power off. You will hear 5 beeps when this happens. The green light will also turn off when the cycle has finished.
-

WE TAKE SERIOUSLY EVERY DETAIL



- ▶ **Orange button**
More obvious and easier to press

Line interface

A standard line interface,
Mainstream easy breathing
machines



- ▶ **Inlet design**



Detection of *Escherichia coli* before and after disinfection



Detection of *Staphylococcus aureus* before and after disinfection



Detection of *Candida albicans* before and after disinfection



Authorized Certification



CE



EMC



FCC